

HEINE OMEGA 600

A quick look at the most important features

We thank you for choosing HEINE OMEGA 600 and wish you a lot of success working with your new instrument.



USB-C Charging socket
Charge with USB-C cord

Charge status indicator

● ● ●
66–100 %

● ● ●
33–66 %

● ● ●
10–33 %

● ● * flashes
<10 %

Brightness Control
On/off, stepless adjustment of the light intensity from 3–100 %, and visionBOOST from 100–245 % for up to 20 % better view in media opacities.
Can be positioned for right and left-handed users

Filter selection lever



Red-free, yellow and cobalt blue filters

For more information or assistance in setting up your instrument to your personal needs, contact us today.



HEINE OMEGA 600



A quick look at the most important features



Height adjustment

Adjustment lever of the optical unit
For height and distance

Aperture selection lever



HEINE Diffusor, small, medium and large spot

Illumination height adjustment

Stereoscopic adjustment lever
Best possible 3D imaging for any pupil size and peripheral views



Flip-up function of the optics unit
For more convenient patient interactions



Eyepieces
To best accommodate your view, exchange the initially mounted eyepieces +2D with the neutral lenses OD (supplied with the instrument)



Teaching mirror TM1



Width adjustment



Dust cover for front glass